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- 1. Description**  
The Optimos™ Stone Basket is a device that is designed to remove stones and/or foreign bodies. It consists of a handle, shaft, connector and sheath. This device is available in a variety of lengths and designs for basket configuration.
- 2. Indications for Use**  
The Optimos™ Stone Basket is used for the endoscopic removal of biliary stones or foreign bodies.  
  - **Patient Population**  
Not specialized for a particular population, but applicable to all patients over 18 years of age; however, the decision rests with the physicians
- 3. Performance and Clinical Benefits**
  - Performance: Bile stone extraction
  - Clinical benefit: Stone clearance
- 4. Contraindications**  
Contraindications for the Optimos™ Stone Basket include, but are not limited to:
  - Any use other than those specifically outlined under Indications for Use.
  - Patients for whom endoscopic techniques are contraindicated
  - Patient with bleeding disorder
  - Patient with ascites
- 5. Potential Complications**  
Potential complications associated with the use of the Optimos™ Stone Basket may include, but are not limited to:
  - **Gastrointestinal endoscopy and ERCP procedure**  
Pancreatitis, cholangitis, sepsis, perforation, hemorrhage, aspiration, fever, infection, hypotension, allergic reaction, respiratory depression or arrest, cardiac arrhythmia or arrest.
  - **Basket extraction**  
Impaction of object, aspiration of foreign body, localized inflammation, pressure necrosis
- 6. Precautions**
  - Identify the location and size of the stones to be retrieved using an endoscope and choose the proper model.
  - Check the state of packaging and sterilization and the expiration date of the product.
  - Make sure you completely understand how to use the product.
  - Check the shape of basket carefully by moving the basket back and forth.
  - If the wire of the basket is bent or twisted when the stone basket has been spread, stop using the product.
- 7. Cautions**
  - ⚠ If any serious incident occurs in relation to this device, the user and/or patient should report to the competent authority of the Member State in which the user and/or patient is established and to the manufacturer.
- 7. Cautions**
  - When this product is inserted using an endoscope, the endoscope should have an accessory channel of an appropriate size for the product. (Refer to the attached label.)
  - The Optimos™ Stone Basket is supplied sterile. Do not use if the packaging is opened or damaged.
  - Do not spread the basket when it is not located the appropriate position
  - Do not use the device where there is resistance.
  - Procedures using the product must be performed only by appropriately trained doctors.
  - Do not use to remove multiple stones at once.
- 8. Warnings**
  - Those who experience allergic reactions to drugs may also show allergic reactions to this operation.
  - This product is disposable.
  - Do not re-sterilize or reuse this product.
  - Do not use this product when the patient has a severe fever, hemorrhaging, or difficulty in breathing.
  - This product is supplied in sterilized packages; do not use this product if its internal package has been damaged due to impact, etc.
  - Use only parts of this product whose shelf life has not expired
- 9. Instructions for Use**
  - (1) Determine where the stone is located using endoscopic and fluoroscopic guidance.
  - (2) Insert the Optimos™ Stone Basket through the endoscope to where the stone is located (check the condition of basket before insertion). When using a contrast agent, inject it after advancing the sheath beyond the stone.
  - (3) Advance the basket beyond the stone, spread the basket by pushing the handle, grasp the stones, and gently retrieve the basket with the stones.
  - (4) Remove the stones from the distal part first when retrieving the multiple stones.
  - (5) Do not pull out the shaft anymore, if the basket is impacted with the stones.
  - (6) Remove the handle and sheath from shaft and insert the lithotripsy through the shaft.
  - (7) Operate the lithotripsy until the stones are crushed or the wires of basket are separated. If passage is still restricted, surgical intervention may be necessary.
  - (8) Remove the product from endoscope, and dispose of the product in accordance with the disposal procedure separately determined by the hospital.
- 10. Handling / Storage**  
The expiration date of this product is three years. Do not use any piece of this product if it has expired. Store this product in a dry, clean place at the temperature of 15°C~30°C; keep the product away from direct sunlight.

### ⚠ Reuse Precaution Statement

This product has been sterilized using ethylene oxide (E.O.) gas.  
Do not use if the sterile package is opened or damaged. The company is not responsible for re-sterilized products.

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**TAEWOONG MEDICAL CO., LTD**